

Talking Switch

Supplies:

Voice Recording Photo Frame

Hot glue stick

Velcro

Hot glue gun (optional)

Instructions:

1. Cut a slice of glue stick about 1/4" thick.
2. Align the piece of hot glue stick so that when the frame is closed the glue stick will activate the play button.



3. Adhere the glue stick to the top of the lid with Velcro or with hot glue. If you use Velcro you may detach it easier later.



4. Use a Velcro strip to secure the lid so that it doesn't flip up.
5. Attach a piece of loop Velcro to top of lid so that you can attach a photo, communication symbol or word that represents the recorded message.

